**MOPEDIA**

MA RP520-RP685B-RP690B-RP691 02 A\_07-2018

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---

**ROLLATORS  
INSTRUCTION MANUAL**

## INDICE

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## Class Medical Device according to directive 93/42/EEC and subsequent amendments

### 1. Codes

RP520	4-wheel aluminum rollator with seat ATLANTE
RP685B	3-wheel steel rollator CRYO
RP690B	4-wheel steel rollator - blue color DYONE
RP691	4-wheel steel rollator with antibrachium rest ERA

### 2. Introduction

Thank you for purchasing the rollator, **MOPEDIA** by MORETTI S.p.A., realized to help people with movement difficulty inward and outward, during voyages or in everyday life. The production has been made according to the highest quality and safety standards in accordance with European Directive on Medical devices (93/42/EEC).

This instruction manual gives you suggestion for a correct use and for your safety. We recommend you to read carefully this instruction manual before using. In case of doubts, please contact your dealer.

**Note:** Check all parts to verify it won't be any shipping damage. In case of shipping damages, do not use. Contact the dealer for further instructions.

### 3. Intended use

This rollator fits for people with movement problems. Can be used as assistance to facilitate ambulation or even servo-assisted ambulatory. Anyway, it can't be used by people with evident malfunction of the hand or the arm, even by people with balance problems. The rollator can be used inward and outward. We recommend using on flat surface where the wheels can reach the land. It can't be used on rock ground or on slanted area as the rollator can be unstable. The device is not intended to carry people or things.

#### **WARNING!**

Do not use the product for a purpose not indicated in this manual. Moretti Spa declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product. The manufacturer can modify the device or the instruction manual without prior notification.

### 4. Declaration of conformity

Moretti S.p.A. declare under sole responsibility that the product made and traded by Moretti S.p.A. and belonging to rollator range are in accordance with the European Directive on Medical devices (93/42/EEC). Moretti S.p.A. guarantees and declared under sole responsibility what follow:

1. The devices under discussion satisfy the essential requisites requested by the 1° enclosure directive 93/42/CEE as laid down by the VII° enclosure of the above mentioned Directive.
2. The devices under discussion ARE NOT MEASURING INSTRUMENTS.
3. The devices under discussion ARE NOT MADE FOR CLINICAL PROBES.
4. The devices under discussion are traded with NON STERILE BOX.
5. The devices under discussion belong to class I°.
6. IT IS NOT ALLOWED to use and install devices except for the use provided by Moretti S.p.A. Moretti S.p.A. places technical documentation at relevant authorities in order to prove the conformity of 92/42/EEC device.

7. Moretti S.p.A maintains and provides for authorities, the technical documentation proving the accordance to European Directive on Medical devices (93/42/EEC).

## 5. Warnings

### 5.1 General warnings

- Do NOT install or use the device before reading and understanding this manual; If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional dealer or technical personnel before attempting to install this equipment to avoid injure or damage;
- The rollator should only be used by patients with a suitable motor capacity which ensures their use without risks;
- Advise with physician or therapist to determinate the correct adjustment and the correct use;
- Physician or therapist should contribute to height adjustment for the maximum support, for a correct brake activation and for a general assembly;
- The rollators are not intended to be used as self-propelled devices.

### 5.2 Warnings for use

- Insure the brake cable to be correctly set;
- Verify the correct operation of the brake after every height adjustment to avoid a free flexion of the cable;
- Breaking this simple rules can cause personal damages;
- The rollator can be used only on a hard, flat surface. Do not use the rollator to climb the stairs
- Leaning always on both dumbbells to maintain balance;
- Pay particular attention to wet, icy or snowy;
- Do not overload the rollator;
- Do not use the rollator seat (where present) to carry persons or property, the seat is for the user's session with the parking brake engaged;
- All wheels must be in contact with the floor at any time during use to ensure the proper balancing of the rollator and maximum safety;
- The brake must be engaged before using the seat.

**Note** When the rollator is used in steady position, the handbrake must be locked

### 5.3 Stability warnings

- The rollator can provide ambulatory assistance to an individual weighing up to 136 kg, INCLUDING the loading bag;
- The rollator bag has a weight limitation of 5 kg.

### 5.4 Assembling & folding warnings

- Verify that both handle support tubes are correctly inserted in the frame and secured with locking knobs locked into the holder with its main wing nuts and knobs without movement;
- Both handle tubes are securely locked in the main support part with wing nuts and knobs without any movements;
- Do not hanging anything on the rollator frame to avoid distortions of the tubes when the rollator is open and damage the stability. Injury may also be caused by distortions;
- Take care to entrapment of fingers and / or clothing during the opening and closing.

## 6. Before use

Before using the device, always check:

1. That all parts of the rollator are locked in a secure manner;
2. Check that all screws are tight;
3. Make sure that the seat is properly locked  
(if that 'not press further with the power seat bottom frame);
4. Check that the locking of the seat is correctly inserted;
5. Check the correct operation of the brake and the regular brake.

## 7. Assembling instruction

### 7.1 Before assembling the product

Unpack the product and check that no damage has occurred during shipping. In the event when the rollator is damaged, please contact your nearest dealer.

## 8. Assembly

### 8.1 Open the rollator

RP520 - RP690B - RP691

Open the rollator pulling the front part of the frame by leveraging on the legs

RP685B

Open the rollator frame and push the reinforcement bar in the front part down.

### 8.2 Pushing handles assembly

Place the handle tubes in the frame. Turn the locking knobs until they are locked.

### 8.3 Adjustment of the handles

Adjust the height of the handles using the side knobs. Unscrew the knob, place the handles to the desired height, turn the knob clockwise to tighten and fasten it again. Repeat for the other handle and make sure that both handles are positioned at the same height.

**WARNING!** Make sure that the handle is in line with the direction of movement of the rollator. If the handle does not stay straight, there is a risk that the rollator wobbling or tipping over, especially if you push it harder.

### 8.4 Folding the rollator

RP520 - RP690B - RP691

Make lever on the lateral bars and bring back the front of the rollator.

RP685B

Remove the tray, basket, bag, pull the lever reinforced inwards and fold the rollator.

### 8.5 Tray (only for RP685B and RP690B)

The tray serves only to support small objects.

**WARNING!** Do not place hot objects, sharp or heavy on the tray. To fix the tray to the frame, insert the support that are on the back of the tray in the corresponding slots on the seat. (only for RP690B model)

### 8.6 Basket (for RP685B - RP690B - RP691)

The shopping basket is hung to the rollator frame.

**WARNING!** Use the basket for small items only. Do not overload the basket to prevent imbalances and the risk of overturning

### 8.7 Bag (only for RP685B)

The bag is attached to the frame of the rollator through the clips.

**WARNING!** Maximum capacity for bag 5 kg. Use the bag for small items only. Do not overload the bag to prevent imbalances and the risk of overturning

### 8.8 Seat (only for RP520 - RP690B - RP691)

Before sitting, place the rollator on a flat and solid surface and then lock the brakes. Make sure the brakes are securely locked and that the rollator is in a stable position.

**WARNING!** Sit down distributing your weight on the seat. Do not sit by pushing only on one side.

## 9. How to use

- Start pushing when you are in an upright position with both hands on the handle;
- Make sure your hands are dry before using the rollator to avoid the risk of slipping;
- Walking maintaining an upright position with his eyes facing forward and not down;
- Do not push the rollator too far away from the body, this may cause falls and loads inadequate;
- Do not use the rollator to overcome steps higher than 20mm;
- When using outdoors, prefer the sidewalks properly prepared in accordance regulations for the removal of architectural barriers;

## 10. Brakes

### 10.1 To brake

To brake the rollator pull the brake lever with your fingers upwards until the rollator slows down or stops. To continue to move the rollator, release the brake handles.

### 10.2 To lock the wheels

Press down on the brake lever with the palm of your hand, when you hear a "click" wheels are locked.

**WARNING!** When the user is stationary, the brakes must be locked

### 10.3 To unlock the wheels

Pulling on both brake levers until they are unlocked

## 11. Care and maintenance

The devices **MOPEDIA** by Moretti SpA on the market are carefully checked and carry the CE mark. For the safety of the patient and the doctor is recommended to be checked by the

manufacturer or by an approved laboratory, at least every 2 years, the suitability for use of your rollator. If repairs are to be used only original spare parts and accessories. The useful life of the product depends on the use, maintenance, storage and cleaning.

## 12. Cleaning

To clean parts, use only a damp cloth with a neutral detergent, not use cleaners containing solvents and metal brushes or abrasive or sharp objects, may damage the product.

## 13. Wheel care

1. Clean the wheels with warm water and mild detergent. Wipe dry with a clean cloth.
2. Periodically lubricate the wheels.

## 14. Technical features

Series ATLANTE										
Product code	RP520									
Seat	38x34 cm									
Frame	Aluminum									
Weight and dimensions										
Measures in CM										
RP520	69	81 96	56	61	44	36	7,4	20	20	136 kg
Series CRYO										
Product code	RP685B - Blue									
Seat	Without seat									
Frame	Steel									
Weight and dimensions										
Measures in CM										
RP685B	56	81 96	66	33	28	7,3	20	20	136 kg	
Series DYONE										
Product code	RP690B - Blu									
Seat	38x16 cm									
Frame	Steel									
Weight and dimensions										
Measures in CM										
RP690B	64	75 98	61	60	44	31	10	20	20	136 kg

Series ERA										
Product code	RP691									
Seat	38x16 cm									
Frame	Steel									
Weight and dimensions										
Measures in CM										
RP691	64	100 122	61	60	44	31	10	20	20	136 kg

## 15. Spare parts / accessories

For spare parts and accessories please refer to the Moretti S.p.A. master catalogue

## 16. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to.

The correct intended use is specified in this manual.

Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use.

Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 16.1 Repairing

**Warranty repair:** If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

**Repair not cover from warranty:** A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

**Non-defective device:** The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

### 16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months as from the receiving date.

### 16.3 Exempt clauses

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

## WARRANTY CERTIFICATE

Product \_\_\_\_\_

Purchasing date \_\_\_\_\_

Authorized dealer \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

Sold to \_\_\_\_\_

Street \_\_\_\_\_ Place \_\_\_\_\_

MORETTI S.p.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

**MORETTI S.P.A.**  
Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11  
Fax. +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)