DOLOMITE® Futura™ DOLOMITE® Symphony™

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This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

©2018 Invacare®Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future
reference.

1 General	4
1.1 About this manual	4
1.2 Intended use	4
1.3 Service Life	5
1.4 Scope of delivery	5
1.5 Warranty	5
2 Safety	6
2.1 General Warnings	6
2.2 Labels and symbols on the product	6
3 Usage	8
3.1 Before use	8
3.2 Folding and unfolding the walking aid	8
3.3 Using the hand brakes	9
3.4 Adjusting the handle height	10
3.5 Walking	11
3.6 Sitting down on a chair	11
3.7 Rising to a standing position	11
3.8 Sitting on the walking aid	12
3.9 Accessories	12
4 Maintenance and care	13
4.1 Maintenance	13
4.2 Cleaning	13
5 After Use	14
5.1 Disposal	14
5.2 Re-use	14
6 Technical Data	15
6.1 Dimensions and weights	15

1 General

1.1 About this manual

This User Manual contains important information about your product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety instructions.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

1.2 Intended use

The walking aid is intended for support and improvement of a limited walking ability. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others.

The walking aid is intended to support a person during walking and may also serve as a resting seat. Furthermore the walking aid is intended to carry a load, either in a specially designed basket or on a tray.

The walking aid is not intended for use by persons with severe equilibrium disorders or functional perceptual disorders.

Permissible operating conditions and locations

The walking aid is only intended for use indoors, in living or public areas, or outdoors. The walking aid is not to be used under water in e.g. pools or open-air bath, neither in hot areas as e.g. saunas.

1.3 Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

1.4 Scope of delivery

- Walking aid, fully assembled.
- User manual

Accessories, actually assembled to your walking aid, are depending on the configuration and might differ per region.

1.5 Warranty

We supply a two year warranty from the date of delivery. Deviations may occur due to local agreements. Damage caused by normal wear to e.g. wheels, handles and brakes

or damage caused by abnormal use (e.g. exceeded max. user weight) is not covered by the warranty. Maintenance and operating instructions must be followed for the warranty to be valid.

2 Safety

2.1 General Warnings

WARNING!

Risk of falling

- Exercise caution when walking on sloping / slippery ground and on uneven surfaces.
- Do not use the rollator as a wheelchair.
- Be careful when walking backwards over obstacles.
- Never sit and push with your feet.
- Do not go up or down stairs with the rollator.
- Do not use the rollator on escalators.
- Do not under any circumstances stand on the rollator.
- Be careful when using long skirts, the clothes might get stuck in the wheels.
- Loads must only be transported in the basket or on the tray. Be careful when there is a heavy load in the basket (for the max. load in the basket refer to the chapter “Technical data”).

WARNING!

Risk of injury

- Do not let children play with the rollator.

CAUTION!

Risk of first degree burns

- Be careful if the push handles or the seat are exposed to extreme temperatures (above 41°C / 106 °F).


IMPORTANT!

Risk of corrosion

- The rollator must not be stored in wet or humid environments.

The person who prescribes the product or who is otherwise responsible for the user must ensure that the product is suitable for the user. The user shall be instructed how to use the product. This includes the use of the features and the restrictions in use. The user shall also be notified of the warnings for use given in the user manual.

2.2 Labels and symbols on the product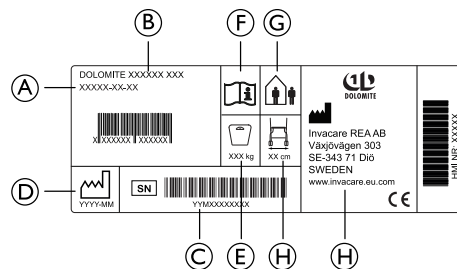

The label contains all important technical information regarding your walking aid.

- Ⓐ Article number
- Ⓑ Product name
- Ⓒ Serial number
- Ⓓ Date of manufacture
- Ⓔ Max. user weight
- Ⓕ Refer to user manual
- Ⓖ In- and outdoor walking aid
- Ⓗ Max. width of walking aid
- Ⓘ Manufacturer's name.

3 Usage

3.1 Before use

We do not recommend that the product is issued to anyone without carrying out a full assessment of their specific need first. The user's remaining walking ability needs to be sufficient for leaving the home, for usual daily activities and for participating in social life without endangering themselves or others. It is important that the user is given the necessary support by the product as prescribed by a qualified professional. This is both for the safety of the user and the rehabilitation process.


3.2 Folding and unfolding the walking aid

WARNING!
Risk of injury

– Never lift the rollator by the folding handle.

Unfolding to operating position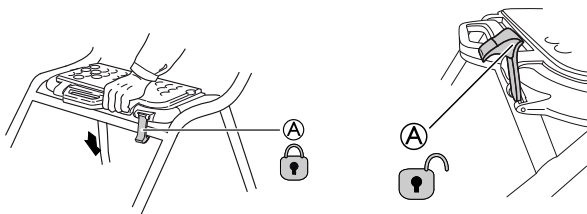

1. Hold the walking aid with the push handles and make sure that all four wheels are stable on the ground.
2. Press both push handles downwards. Make sure that the seat plate is folded completely down (see picture).
3. Push down the safety latch (A) in the locked position.


WARNING!

– Before using the product, always check that the safety latch (A) is in locked position indicated by a clear audible “Click”.

Flip up the seat plate


The seat can be folded up to provide a larger walking space, and in order to increase accessibility.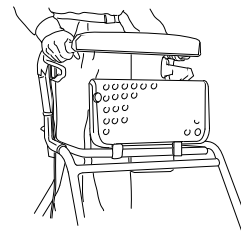

IMPORTANT!


The walking aid must be fully unfolded and locked with the folding handle before using the product.
– The folding handle locks when a clicking sounds is heard.

Folding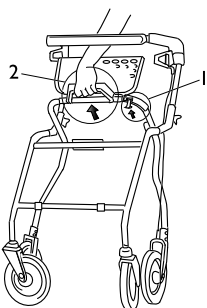


1. Lift up/loosen the safety latch.
2. Lift the handles in order to fold the walking aid into folded position.

3.3 Using the hand brakes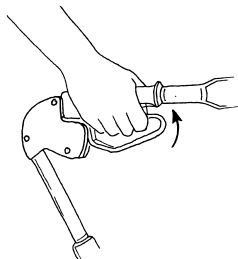

1. Pull both brakes upwards to brake.

Using the parking brake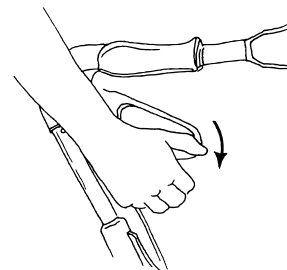

1. To apply the parking brakes, press the brake handles carefully downwards.
An audible sound indicates that the brake is in the correct position.


IMPORTANT!

- Make sure that both brakes are put into parking position.

2. To release the brakes, pull the brake handles upwards.

Adjusting the braking effect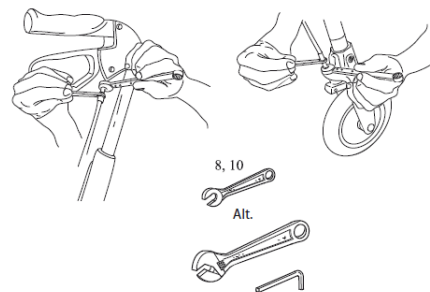

For desirable braking strength, adjust the brake wire.

Suitable tools: 8 mm and 10 mm spanner (not included).

 The wire can be adjusted in two places - above under the brake handle or below above the brake clamp. The brake handle must be in neutral position.


1. Loosen the locking nut and turn the hexagon nipple counter clockwise in order to increase the braking effect. The distance between the brake clamp and the wheel should be 2 mm.
2. Tighten the locking nut again to fixate the new adjustment.
3. Check the braking effect by engaging the parking brake and check that the wheel can't rotate.

WARNING!

– Make sure to tighten the lock nuts after adjustment.

3.4 Adjusting the handle height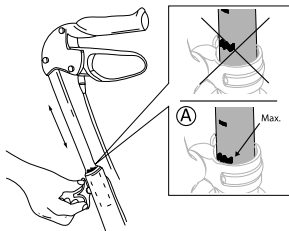

1. Stand behind the walking aid with flat shoulders.
2. Loosen the knob.

3. Adjust the handle to suitable height.

The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.

4. Retighten the knob.

WARNING!

Risk of falling

– Ensure that the handles never been set above their highest position, indicated by the marking **A** on the tubes.

5. Repeat the procedure on the other side.


If the user unconsciously tends to adjust the height adjustment handles, for safety reason the knobs should be replaced by locking screws (available as accessory).

IMPORTANT!

– For optimum performance ensure that handles are in the allowed adjustment area at all times. The arrow, which is placed on the tubes on the inside of the walking aid, pointing on the thin inner vertical line indicates maximum allowed outer handle position.

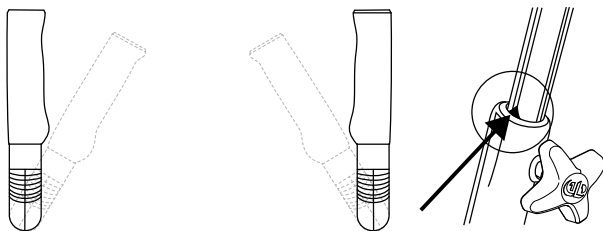
3.5 Walking


WARNING! **Risk of falling**

- Be careful if the wheels stop against an obstacle.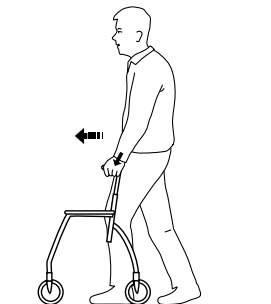

1. Place the walking aid slightly in front of you and take a step towards it while using the hand grips for support.

3.6 Sitting down on a chair

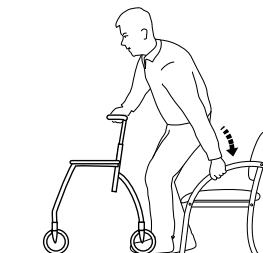
1. Stand in front of the chair with the walking aid, the back of your legs should touch the chair.
2. Place your hands on the arms of the chair, lean forward slightly and sit down carefully.

3.7 Rising to a standing position

WARNING! **Risk of tipping**

- Never use the walking aid to pull yourself up from a sitting position.
- The parking brakes must be in braked position when the walking aid is used as a support when sitting down or getting up, from a chair for example.

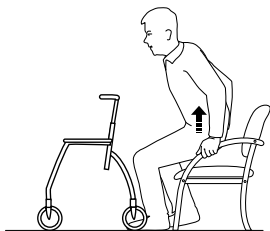1. Place the walking aid in front of you, use the armrests of the chair as support and rise carefully to a standing position.
2. Once stable in a standing position, transfer hands to the hand grips of the walking aid.

3.8 Sitting on the walking aid

WARNING!

Risk of falling

- Use the walking aid for stationary seating only. Both brakes must be in parking position when the walking aid is used as a seat.
- Never sit and push with your feet.
 - Never sit on the walking aid on sloping ground.

3.9 Accessories

Accessories are not standard on all models. For a list of available accessories, contact your authorized dealer or the Invacare representative (see rear page of this manual).

Basket

Mount the basket by hanging it with the hooks on the front frame.

Tray

Secure the tray by using the plastic plugs.

4 Maintenance and care

4.1 Maintenance

 Maintenance should be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Ensure that screws and adjustments are properly tightened.
- Never use a defective walking aid. Do not attempt repairs yourself!
- If hand grips are loose do not use the walking aid. Change hand grips according to the service instructions which can be obtained from your Invacare representative.
- Verify brake operation. If brake adjustment is required, see chapter 3 *Usage, page 8* in this user manual.
- Periodically inspect the castors tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

4.2 Cleaning

The product can be cleaned and disinfected using commercially available detergents.

IMPORTANT

- Cleaning must be done at least once a year.
- Do not use an abrasive detergent cleaner.
- Do not use a steam cleaner.
- Never use acids, alkalies or solvents.

- The product can be washed in a washing machine or by hand.
- The washing and drying temperature may not be higher than 60° C.
- High-pressure cleaner or ordinary jet of water may be used. However, do not aim the jet towards bearings, draining holes or labels.
- Clean with damp cloth and mild detergents.
- Clean the wheels with a brush (no wire brush!).
- After washing, wipe the product dry.

Disinfection

Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

5 After Use

5.1 Disposal

When the product can no longer be used, it and its accessories as well as the packaging material should be sorted in the correct way.

The individual parts can be separated and handled according to their material.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.2 Re-use

This product is suitable for re-use after the following has been performed:

- Inspection
- Cleaning

For details, see chapter 4 *Maintenance and care*, page 13.

**WARNING!****Risk of injury**


- Always inspect the product according to the service instructions before transferring the product to a different user.
- If any damage is found do not use the product.

The user manual belongs to the product. Make sure that the user manual will be handed over with the product.

6 Technical Data

6.1 Dimensions and weights

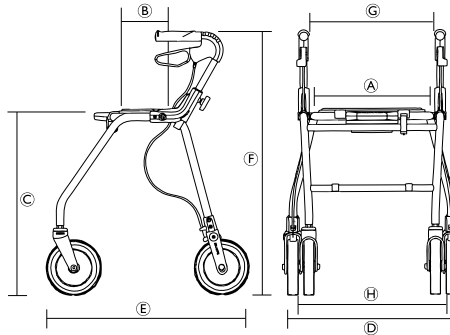[mm]	Symphony™450 Futura™450	Symphony™520 Futura™520	Symphony™600 Futura™600	Futura™680
Seat width (A)	360			
Seat depth (B)	180			
Seat height (C)	475	545	620	705
Total width (D)	600	600	600	640
Total depth (E)	590	660	700	780
Total height (Handle height) (F)	640 - 790	710 - 850	790 - 930	860 - 1015
Distance between handles (G)	420			

[mm]	Symphony™450 Futura™450	Symphony™520 Futura™520	Symphony™600 Futura™600	Futura™680
Distance between rear wheels Ⓜ	460	470	470	500
Front/rear wheel ¹ diameter	200			

[kg]	Symphony™			Futura™			
	450	520	600	450	520	600	680
Weight ²	6,7	6,8	7,0	7,6	7,8	8,0	8.2
Max. user weight ³	100			150 for walking aids produced 2011–03 or later *			
Max. load basket	7	9		7	9		
Max. load tray	5						

 * For walking aids produced 2011–02 or earlier; See label on product

¹ Front wheels are steering wheels. Rear wheels are provided with a brake system for dosed breaking and for locking the wheel.

² Total weight excluding equipment.

³ This weight limitation includes the weight of the user whether standing or sitting and the content of the basket.

6.2 Material

	Symphony™450 / 520 / 600	Futura™450 / 520 / 600 / 680
Frame	Polyester powder coated aluminium	Polyester (PES) powder coated steel
Handle bars	Aluminium	
Handgrips	Thermoplastic Elastomers (TPE)	

	Symphony™450 / 520 / 600	Futura™450 / 520 / 600 / 680
Wheels	Polypropylene (PP) with treads of Thermoplastic Elastomers (TPE)	
Plastic parts	Material according to marking; Polyamide (PA), Polypropylene (PP), Polyethylene (PE)	
Screws	Steel, rust protected (zinc plated)	

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Invacare Austria GmbH
Invacare Rea AB