READ THE MANUAL BEFORE OPERATING THIS PRODUCT

COMPEX ®

FIXX™ 2.0

Massager

PRODUCT DESCRIPTION

The Compex® Fixx™ 2.0 massager helps warm up muscles before a workout, and relieve stiffness after a hard session. Five, powerful intensity settings, paired with a high stall force, will give the intense pressure needed to relieve the toughest spots safely and comfortably. Interchangeable tips (included) and a rotating head help tailor your massage to the specific needs of different muscle groups. Removable and rechargeable batteries, a compact design, and quiet motor help provide relief anywhere and everywhere.

INSTRUCTIONS

1. Before use, make sure Fixx™ 2.0 massager is off. Then, place the ball on to the metal shaft with slight pressure until the shaft stops inside the ball. Turn clockwise until ball is secured in place. The ball must be connected prior to operating the Fixx™ 2.0 massager.
2. Fixx™ 2.0 massager turns on when battery is inserted into massager. Making sure Fixx™ massager switch is in the OFF position, slide battery into battery slot by pushing battery in until you hear the tabs on each side click into place.
3. To turn on, slide the switch on the top handle forward.
4. To switch to the desirable massage speed level, rotate the speed dial knob clockwise to cycle from one level to the next. The speed level indicator light will show the speed level you are on.
5. To rotate the arm, press the button on the circular housing. Arm can lock into 3 different positions (0, 45, 90 degrees).
6. To remove battery, push the release tab and then pull the battery from the massager.
7. To remove ball, turn Fixx™ 2.0 massager off and turn ball counterclockwise and pull ball away from the shaft. Use antibacterial or alcohol wipes to clean ball between uses.

- A On/Off
- B Speed Dial Knob
- C Speed Level Indicator
- D Battery Level Indicator

BATTERY LEVEL

100%: Indicating lights in white

75%: Indicating lights in grey

50%: Indicating lights in grey

25%: Indicating lights in red

CHARGING INSTRUCTIONS

1. Connect the prong to the adapter and twist clockwise to lock into place. To release prongs from the adapter, press the button on the bottom left and twist counterclockwise the prongs at the same time.
2. To remove battery, push the release tab and then pull the battery from the massager.
3. Insert battery onto the charger base while connecting the cable to the back of it. Then plug the adapter into the proper rated outlet.
4. LED lights illuminate to show battery level and indicate active charging. Red – Charging/Green -- Fully Charged
5. Charging time: 1.5-2 hours

MAINTENANCE, CLEANING & STORAGE

- Wipe down with damp towel to clean device, then dry.
- Wipe ball and/or tips and grip with anti-bacterial wipes to prevent any bacteria contamination.
- Remove battery and place in case for storage or travel.

TROUBLESHOOTING

- If Fixx™ 2.0 Massager will not power on, remove battery and place on charger until battery is charged.
- If ball will not remove from shaft, turn counterclockwise to remove.
- If battery will not charge, check connections of power cord connected to battery charger or wall plug in power outlet.
- If arm won't articulate, press button on the hinging head and move it in opposite direction to release and move to intended position.

RECOMMENDED USE POSITIONS

Visit compex.com to see recommended use positions.

SAFETY WARNINGS

THIS PRODUCT MUST BE USED IN ACCORDANCE WITH THE FOLLOWING INSTRUCTIONS:

- Used by and/or administered by adults only.
- Stop immediately if you feel any pain or discomfort.
- Avoid contact with head and/or other bony areas.
- Use on body parts for no more than 1 minute per specific location. Keep moving over muscles and avoid bony areas.
- Avoid putting fingers, hands and/or hair near the ball/shaft section of Fixx™ 2.0 massager.
- Do not drop Fixx™ 2.0 massager.
- This appliance is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- The appliance is only to be used with the power supply unit<model NO. ICP20-200-1000D> provided with the appliance.
- WARNING: For the purposes of recharging the battery, only use the detachable supply unit provided with this appliance.

CONTRAINDICATIONS

PLEASE DO NOT USE FIXX™ 2.0 MASSAGER WITHOUT RECEIVING APPROVAL FROM YOUR DOCTOR IF THE FOLLOWING CONDITIONS APPLY:

- Recent surgeries of any kind
- conditions and/or wounds affected by vibrating massagers
- Herniated disks
- Pregnancy
- Diabetes/neuropathy or retinal damage
- Wear of pacemakers
- Injuries affected by vibrating massagers
- Epilepsy or migraines
- Disc herniation
- Spondylolisthesis
- Vertebral detachment or spine joint rigidity
- Recent replacement of joints or IUD
- Metal pins or metal plates
- Any health-related diseases

WARNINGS

- Do not use if injured.
- Do not immerse in water, sand and/or loose debris that might get into motor housing vent.
- Do not remove screws or attempt to disassemble.

ADVERSE REACTIONS

You may experience skin irritation and mild bruising if Fixx™ 2.0 is used in one spot for more than 60 seconds or over bony areas. Adverse reactions may be experienced by people recovering from surgery, skin conditions, wounds, nerve conditions, broken bones and/or contusions. To avoid reactions, users should consult doctor prior to using Fixx™ 2.0.

DECLARATION OF CONFORMITY

FCC PART 15B Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

PERFORMANCE SPECIFICATIONS

BASIC SPECIFICATION		SPEED 1	SPEED 2	SPEED 3	SPEED 4	SPEED 5	REMARK
Noise(dB)		58-60	61-63	64-67	68-71	72-74	Condition: 30cm distance
Percussions per minute (PPM)		1000	1500	2000	2500	3000	
Hertz (Hz)		16.7	25.0	33.3	41.7	50	
Power consumption testing	Voltage(V)	16.2	15.8	15.7	15.5	15.4	
	Current(A)	0.47	0.8	1.042	1.5	1.9	
	Power(W)	7.6	12.6	16.4	23.3	29.3	
Charge and Discharge test	Charge time	1.5-2 Hrs					
	Discharge time					1-1.5 Hrs	
Adapter	100-240V-50/60Hz 0.6A/ OUTUT:20V 1A 20W(with International plugs ETL DC5.5 male end, wire length 1.5M Model NO: ICP20-200-1000D						

BATTERY AND CHARGER INFORMATION

- Battery Charger: Input: 20VDC 1A. Output: 16.8VDC – 1A Model NO: LM-CE03
- Rechargeable Li-Polymer battery 1800 mAh 10A, 5C series parallel combination(4 cells)

HANDLING BATTERIES

This product contains a battery. Read the following warning and follow the Battery Safety and Battery Disposal instructions below.

SAFETY WARNING

There is danger if the battery is mishandled or incorrectly replaced. Replace only with the same type of battery associated with this product. Do not disassemble it or attempt to recharge it outside the system. Do not crush, puncture, dispose of in fire, short the external contacts, or expose to water or other liquids. Dispose of the battery in accordance with local regulations and instructions.

BATTERY SAFETY

- Insert batteries correctly. There may be a risk of explosion if the batteries are incorrectly inserted.
- Please follow instructions provided for charging batteries.
- Replace batteries with the same type of battery associated with this product.
- Do not expose batteries to excessive heat (such as sunlight or fire).
- Storage environment: -10°C ~ +45°C.
- It is recommended to charge the battery every three months at least.

BATTERY DISPOSAL

- The batteries may contain substances that could be harmful to the environment.
- Recycle or dispose of batteries in accordance with local/national disposal and recycling regulations.

WHAT TO DO WITH LEAKED BATTERIES

Remove the battery immediately, but proceed with caution: work in a well-ventilated area, away from heat sources. Wear protective eye-wear and gloves if possible to avoid contact with the residue. After removing the leaked batteries, clean out the battery compartment with a cotton swab or an old toothbrush. Put the batteries in a plastic bag and recycle.

Be sure to avoid contact with the discharge at all times. Always wash your hands thoroughly with soap and water afterwards. In case of eye contact, seek medical attention immediately.

WARRANTY

This device carries a limited warranty of two years from the date of delivery. The warranty applies to the device only, the accessories and the battery are not covered by this warranty.

During the warranty period, defective items will be repaired or replaced at no charge. Any evidence of misuse, abuse, alterations, or externally caused damage may have this warranty invalid.

For more information, please contact the distributor/manufacturer.

COMPEX FIXX 2.0 MASSAGER

MODEL: INTL-CX202WL01

Powered by Lithium-polymer battery (14.8V , 1800mAh, 26.64Wh)

Rated Power: Max. 31.72W

Use only with <ICP20-200-1000D> supply unit

Made in China

CONTACT INFORMATION

Distributed by DJO France S.A.S
Centre Européen de Frêt, 3 rue de Bethar
64990 Mouguerre France
www.compex.com
Made in China

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Distribuído por DJO France S.A.S