TERMINAL BASIC

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User manual for Terminal Basic

Thank you for choosing this Soehnle Professional product. This product is equipped with all features of the very latest technology and is optimised to make operation as simple as possible. If you have any questions, or if there is a problem with the equipment that cannot be resolved with this user manual, please contact your dealer or service partner, or visit our website at www.soehnle-professional.com.

1. SCALE MODELS

3700	Basic display unit
7801	Portable personal scales
7802	Chair weigher
7871	Comfort invalid chair scales
7808	Multifunction scales / Wheelchair weigher
7858/7859	Animal weighing scales

2. FUNCTIONS

2.1 Start up

Set up the scale at the place of use and ensure that it is placed horizontally on a firm level surface. Connect the power adapter to the scale.

Before the initial start-up of the scale, the battery must first be charged for 5 hours!

2.2 Switching on

Remove any weights from the scale and press the ON/TARA/OFF button.

After the test routine has completed, the display shows a zero.

The scale is now ready for use.

2.3 Weighing

Either stand or sit on the scale.
The weight is displayed on the scale.

2.4 Taring

Place some paper towels or seat covers on the scale and press the ON/TARA/OFF button.

The display shows zero and the letter N for net weight.
Taring can be done over the entire weight range.

2.5 Weighing small children

An adult steps onto the scales.

> The display shows »57.6 kg« for example

Press the ON/TARA/OFF button.

> The display changes to »0.0 kg«

The adult steps off the scale ...

> The display shows »-57.6 kg«

... then steps onto the scale again holding the child.

> The display now indicates the child's weight, for example »5.1 kg«

2.6 Switching Off

Press the ON/TARA/OFF button to switch off the scale.

If the scale is empty and the display indicates zero weight, the scale can be switched off immediately.

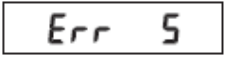
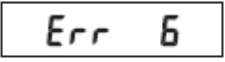
If the scale is loaded, press and hold the ON/TARA/OFF button for 5 seconds.

Note:

If the display indicates »0,0 kg«, the scale switch off automatically after a preset time. In this case, the display can be activated again by pressing the ON/TARA/OFF button.

3. ERRORS – CAUSES AND REMEDIES

DISPLAY	CAUSES	REMEDY
	Underload: only dashes appear at the bottom of the display field.	Switch the display off and on. The zero point is reset automatically.
	Overload: only dashes appear at the top of the display field. Maximum weighing range is exceeded.	Unload the scales, or switch them off and on again. The zero point is reset automatically.
	Scales were not stabilised when switched on.	Check the surface under the scales, and check that the weighing platform is level. Test the scales with a force shunt. If the error persists after readjustment, please contact your service partner.
	Taring not possible.	Stand still, or stabilise the weight on the scales.

If the error cannot be cleared, or other error messages appear, please contact your service partner.

4. GENERAL INFORMATION

4.1 Technical data

The weighing range and scale interval are indicated on the data plate on the scales.

Operating temperature: + 10 °C to + 40 °C

Storage temperature: - 20 °C to + 65 °C

The technical specifications of the power supply are printed on the power adapter.

4.2 Intended use

The BASIC display unit is designed for weighing people. It is not intended for custody transfer applications. The non approved version of the product meets the applicable requirements of EC guidelines 2004/108/EC and 2006/95/EC.

4.3 Safety information

Before starting up the device, please read carefully through the information given in the user manual. It contains important information concerning the installation, correct use and maintenance of the device.

In accordance with EC guideline 2004/108/EC, this device is equipped with interference suppression. However, the display reading can be affected by extreme electrostatic or electromagnetic interference, for example if a radio or a mobile phone is used immediately beside it. The display should return to normal once the source of interference is removed. However, it may be necessary to restart the scale. If there is permanent electrostatic interference in the area, we recommend earthing the platform.

This device is a measuring instrument. Draughts, vibrations, rapid temperature changes and direct exposure to sunlight may affect the weight readings.

The scale conforms to protection class IPX2. Avoid high levels of air humidity, steam, aggressive fluids and heavy soiling.

Improper installation will invalidate the warranty. The electrical connection conditions must be consistent with the rating values printed on the power supply unit. The equipment is designed to be used indoors. Note the permitted ambient temperatures for operation (see Technical Data). This equipment meets the requirements for electromagnetic compatibility. Do not exceed the maximum values specified in the standards. If used in the correct conditions, there is no danger of other equipment being affected by this product.

Floor scales may only be driven on in reverse with wheelchairs.

If you have any problems, please contact your service partner.

Warning: This device must not be modified without the manufacturer's permission. If medical equipment is modified, suitable tests and inspections must be performed to ensure that it can still be used safely.

4.4 Cleaning

Before cleaning, disconnect the equipment from the power supply by pulling out the mains plug. The device should be cleaned only with a moistened cloth. In no circumstances should water be allowed to penetrate the device. Disinfectants should be used only on the foil keypad of the display panel.

The following disinfectants are allowed:

Methylated spirits; isopropanol; 2% Kohrsolin; 1% aqueous Sokrena solution; 5% Sagrotan; 5% Gigasept. Do not spray the equipment or the connecting plug.

4.5 Maintenance and service

Metrological control has been carried out by the manufacturer. This scale does not require regular maintenance. However, it is advisable to check its accuracy at regular intervals. The frequency of these checks depends on the type of use and the condition of the scale. If deviations are found or repairs are necessary, please contact your dealer or the manufacturer's Customer Service.

4.6 Warranty / liability

If the supplied goods contain a defect attributable to the manufacturer, the manufacturer will be entitled to choose either to rectify the defect or to provide a replacement. Replaced parts become the property of the manufacturer. If the defect cannot be rectified or the supply of a replacement is unsuccessful, then the statutory regulations will apply.

The warranty covers a period of **2 years** starting from the date of purchase. **Please keep your invoice as proof of purchase.** If your device needs service, please contact your dealer or the manufacturer's Customer Service.

In particular, liability will not be accepted for any damage resulting from any of the following: Unsuitable or inappropriate use or storage, incorrect assembly or start-up by the customer or a third party, natural wear and tear, modifications or tampering, faulty or careless handling, particularly excessive loads, chemical/electrochemical/electrical influences or dampness, insofar as these cannot be attributed to the manufacturer.

The warranty guaranteeing overall correct function of the equipment will become void if operating, climatic or other influences cause a substantial change in the conditions or the state of the material.

Wear parts (e.g. batteries) are covered by a 6 month warranty.

Please keep the original packaging in case a return shipment is necessary.

4.7 Disposal of batteries

Batteries that contain hazardous substances are identified by a **symbol showing a crossed-out waste bin and should not be disposed of as normal household waste**. As a consumer, you are required by law to return used batteries. Your old batteries can be handed in as special waste at the public recycling points in your community, or anywhere that batteries of a similar kind are sold. These symbols are found on batteries containing hazardous substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury.

4.8 Disposing of the scale

According to current state of knowledge, this device does not contain any substances that are particularly harmful to the environment. It must not be treated as normal waste, but must be taken to a collection point where electric and electronic devices are recycled. You can obtain more information from your municipal authorities, local waste disposal services, or the company from which this product was purchased.

5. WARNINGS

- › In the event of a malfunction, please contact your dealer or the manufacturer. Unauthorised modifications or repairs can damage your scale and will cause the manufacturer's warranty to become void.
- › Display units should be protected from moisture. Liquids (e.g. water) can damage the display unit. Use a dry cloth, e.g. a towel, to wipe the display.
- › Disconnect the power supply before installation, cleaning or maintenance of the device. Otherwise, the device could be damaged.
- › Do not expose the display to direct sunlight. This could cause colour changes.
- › If you are not using the display for long periods, disconnect the power supply, or switch off the display if it is operating on battery power. Recharge the batteries after 4 weeks at the latest, to compensate for self-discharge.
- › Avoid stacking any materials on top of the display, or placing any weights on it. This can damage the display.
- › Place the scale on a solid, stable and level surface: this will ensure that measurements are accurate. Accuracy is affected if the scale are tilted or placed on a carpeted floor.
- › Do not connect the scale to an unstable power source.
- › Use only the original equipment. Use of other brands can cause damage to the display.
- › Floor scales may only be driven on in reverse with wheelchairs.

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Soehnle Industrial Solutions GmbH